



**L&R Manufacturing Company**

577 Elm Street, P.O. Box 607, Kearny, New Jersey 07032-0607 USA

Phone: 201-991-5330, Fax: 201-991-5870

E-mail: [info@LRultrasonics.com](mailto:info@LRultrasonics.com), [www.LRultrasonics.com](http://www.LRultrasonics.com)

## **EARMOLDS**

Ear wax and perspiration soil can be safely and thoroughly cleaned from earmolds allowing better tonal quality with an Ultrasonic Cleaning System from L&R. Simply-

1. Place earmold into basket and immerse into tank containing Earmold Cleaner Concentrate at recommended dilution (See product label).
2. Cavitate 2 - 3 minutes.
3. Rinse thoroughly and dry.

### **EQUIPMENT NEEDED:**

Machines, Accessories\*: Q90, Q140, 2014, or S140 Ultrasonic Cleaning System with Timer, Optional Heater\*\*, Basket, Auxiliary Pan.

Solutions: Earmold Cleaner Concentrate (Product Code 216)

\*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

\*\*Heater not available on Model 2014.