



*because clean matters.*

## PODIATRY

Surgical instruments (scalpels, blades, burs, forceps, clippers, hemostats, rasps, even jet injectors) can be cleaned quickly, safely, economically and thoroughly with Ultrasonic Cleaning Systems from L&R.

Simply-

1. Place instruments in Basket (hinges open) and immerse in tank filled with properly diluted General Medical Cleaner Concentrate (1:10).
2. Cavitate 5 minutes, remove, rinse thoroughly in water.
3. Dry instruments and sterilize.

Note: Jet injectors should be cleaned in a 600 ML Beaker. Place Beaker into Positioning Cover and cavitate for 3 minutes.

### EQUIPMENT NEEDED:

Machines, Accessories\*: Q140 or other Ultrasonic Cleaning System with Timer, Optional Heater\*\*, Basket, Auxiliary Pan, Positioning Cover, 600 ML Beakers.

Solution: General Medical Cleaner Concentrate (PC149).

\*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

