



**L&R Manufacturing Company**

577 Elm Street, P.O. Box 607, Kearny, New Jersey 07032-0607 USA

Phone: 201-991-5330, Fax: 201-991-5870

E-mail: [Info@LRultrasonics.com](mailto:Info@LRultrasonics.com), [www.LRultrasonics.com](http://www.LRultrasonics.com)

## **SURGICAL AND DENTAL INSTRUMENTS**

Dried blood, tissue and protein, as well as common grit and grime are easily and safely cleaned with an L&R Ultrasonic Cleaning System and the proper cleaning solutions.

1. Fill tank to 3/4 full with proper cleaning solution as per instructions on the container.
2. Place instruments into a solid side basket.
3. Place basket into machine and cavitate for 3 - 5 minutes.
4. Rinse thoroughly, proceed to next sterilization procedure and dry.

### **EQUIPMENT NEEDED:**

Machines, Accessories\*: 2014, Q140 or S140 Ultrasonic Cleaning System with Timer, Basket.

Solutions: General Purpose Cleaner Non-ammoniated for Dental instruments (Product Code 228); General Medical Cleaner Concentrate Non-ammoniated (Product Code 149).

\*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.

**APPLICATION BULLETIN #39**