



L&R Manufacturing Company
577 Elm Street, P.O. Box 607, Kearny, New Jersey 07032-0607 USA
Phone: 201-991-5330, Fax: 201-991-5870
E-mail: info@LRultrasonics.com, www.LRultrasonics.com

TATOOING NEEDLES – HORSES

Dried blood and tissue as well as common grit and grime are easily and safely cleaned with L&R Ultrasonic Cleaning Systems and the proper cleaning solutions.

A portable rack for the generators is adapted to run off a car battery. These are carried in a car trunk by the technician who tatoos the lips of race horses.

1. After tatoeing, needles are rinsed in alcohol using the Auxiliary Pan.
2. Place needles into the main tank filled with Aqualar #1 Cleaning Solution Concentrate (properly diluted), and cavitate for 6 minutes.
3. Place needles into tank #2 filled with Cold Disinfecting Solution and cavitate 6 minutes.
4. Remove and dry.

EQUIPMENT NEEDED:

Machines & Accessories*: Two S15OR's mounted in stainless steel frame with Sealing Covers, Auxiliary Pan.

Solutions: Cold Disinfecting Solution; Alcohol. Aqualar #1 Cleaning Solution Concentrate (Product Code 203).

*Recommended Ultrasonic Cleaning system varies as to internal tank dimensions and volume of required cleaning.

APPLICATION BULLETIN #41