



APPLICATION BULLETIN #12

EARMOLDS

Ear wax and perspiration soil can be safely and thoroughly cleaned from earmolds allowing better tonal quality with an Ultrasonic Cleaning System from L&R.

Simply-

1. Place earmold into basket and immerse into tank containing General Purpose Cleaner Non-Ammoniated at recommended dilution (See product label).
2. Cavitate 3-5 minutes.
3. Rinse thoroughly and dry.

EQUIPMENT NEEDED:

Machines, Accessories*: Q90 or Q140 Ultrasonic Cleaning System with Timer, Optional Heater**, Basket.

Solutions: General Purpose Cleaner Non-Ammoniated (PC228)

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.



because clean matters

L&R Manufacturing Company
577 Elm Street, PO Box 607
Kearny, NJ 07032-0607 USA
Tel: 201.991.5330 Fax: 201.991.5870
www.LRultrasonics.com info@LRultrasonics.com