



APPLICATION BULLETIN #42

TATTOOING NEEDLES - HUMANS

Clean oxides and sulphides safely and quickly with L&R Ultrasonic Cleaning Systems and proper cleaning solutions.

Simply-

1. Fill Q140 2/3 full with appropriate dilution of UltraDose Ultrasonic Cleaning Solution or other Solution.
2. Carefully place needle assembly in Basket. Do not allow needle tips to touch tank bottom.
3. Cavitate for 6 minutes.
4. Rinse thoroughly in water.
5. Dry and sterilize.

EQUIPMENT NEEDED:

Machines, Accessories*: Q140 or other Ultrasonic Cleaning System with Timer, Basket.

Solutions: UltraDose Ultrasonic Cleaning Solution (PC012)
UltraDose Germicidal Ultrasonic Cleaning Solution (PC 036)
UltraDose Enzyme Plus Ultrasonic Cleaning Solution (PC 038)

*Recommended Ultrasonic Cleaning System varies as to internal tank dimensions and volume of required cleaning.



because clean matters

L&R Manufacturing Company
577 Elm Street, PO Box 607
Kearny, NJ 07032-0607 USA
Tel: 201.991.5330 Fax: 201.991.5870
www.LRultrasonics.com info@LRultrasonics.com